

ProfiGuard, Prophylaxepaste RDA250

Description

Prophylactic paste for the gentle removal of plaque and discoloration.

Application

After removal of dental calculus place ProfiGuard into a dappen dish or other suitable container. Use a small quantity on a rubber polisher or polishing brush and apply to the area of treatment. Only start handpiece if polisher is in contact with the tooth surface. Polish at low speed and with little pressure. Allow polisher to reach areas below the gingiva. Repeat the treatment with fresh paste until all surfaces are cleaned and polished. Use Becht dental floss to apply the paste to the interproximal spaces. Finish by rinsing thoroughly.

Properties

- Gentle removal of plaque and discoloration
- Sodium-fluoride for effective protection against caries and parodontal diseases
- Mild mint flavor
- RDA 250
- 100% vegan, lactose-free
- gluten and phthalat-free



Composition

- Pumice stone
- amorphous Siliciumdioxide
- Glycerin
- Paraben
- Sodium-fluoride (0,2%)
- flavor-agents

Shelf life

60 Months

Classification

Medical Device, Class I, Rule 5

Form of delivery

REF 480 Tube with 95 g

Manufacturer

Alfred Becht GmbH
Carl-Zeiss-Str. 16
D-77656 Offenburg
Tel.: +49 781 60586-0
Fax.: +49 781 60586-40
www.becht-online.de

Accessories

Cup Brushes, 4 mm, mounted on CA cups

REF 260.21 white, with natural bristles

REF 260.2N white, with nylon filament

REF 260.240N blue, with nylon filament

REF 260.250N pink soft, with nylon filament

REF 260.260N white soft, with nylon filament



Prophy Cups, latex-free

REF 200.1, 200.2, 200.3 Screw-in cups

REF 202.1, 202.2, 202.3 Snap-on cups

REF 204.1, 204.2, 204.3 Mounted on CA

