

## Asphaline temp

### Description

Root filling material for temporary root filling of gangrenous teeth.

### Application

Prepare cavity and root canal, rinse and dry. Rinse out any residual hydrogen peroxide, since it may slightly impair the effect. Hypochlorite solution does not affect use. Select Lentulo and coat with a little paste. Work the paste around slowly into the root canal. Repeat procedure if required. Do not overfill. You can check by X-ray that the filling is complete. Asphaline temp does not set completely, and should be removed after a period of 5 days. With repeated use, an accumulated period of 14 days in the root canal should not be exceeded.

### Properties

- ready-to-use paste
- encourage complete healing of infected areas
- radiopaque
- free of paraformaldehyde and eugenol
- adheres excellent to the dentin
- mischbar mit anderen wasserbasierenden Pasten

### Composition

- Polymer adsorbed iodine
- Zincoxide
- Bariumsulfate

### Form of delivery

REF 550 Tube with 7 g paste

### Accessories

REF 930-933; 937 Paper Points



### Shelf Life

30 Months

### Classification

Medical Device, class IIa, rule 7,  
CE Certificate: G1 17 09 49628 010  
TÜV SÜD Product Service GmbH, München

### Manufacturer

Alfred Becht GmbH  
Carl-Zeiss-Str. 16  
D-77656 Offenburg  
Tel.: +49 781 60586-0  
Fax.: +49 781 60586-40  
Germany  
[www.becht-online.de](http://www.becht-online.de)