

Prophylaxis Brushes

Description

Prophylaxis Brushes are designed to remove dental plaque and to polish tooth surfaces using prophylactic paste. As a result, aesthetically clean tooth surfaces are achieved.

Application

Prophylaxis Brushes are used with prophylactic paste. The Prophy Brushes were mounted on a dental contra-angle handpiece. After applying the paste to the cup or directly to the tooth, the dental contra-angle handpiece is placed on the tooth and is set in rotating motion.

Properties

- White, hard: hard, for removing severe plaque
- Blue: Medium, universal hardness for cleaning and polishing
- Pink: soft, for final polishing of the tooth surface, especially near the gingival sulcus to protect the soft tissue
- White, soft: soft, for final polishing of the tooth surface especially near the gingival sulcus to protect the soft tissue
- Becht Prophylaxis Brushes are intended for single use only. All packaging units are shipped non-sterile.
- Maximum working speed: 2500 rpm



Shelf life

60 Months

Form of delivery

Cup Brushes, mounted on CA cups

REF 260.21 with natural bristles, 12 pieces

REF 260.230 with natural bristles, 100 pieces

REF 260.2N white, with nylon filament, 12 pieces

REF 260.230N white, with nylon filament, 100 pieces

REF 260.240N blue, with nylon filament, 100 pieces

REF 260.260N white soft, with nylon filament, 100 pieces

REF 260.250N pink soft, with nylon filament, 100 pieces

Classification

Medical Device, class IIa, rule 5

CE-Certificate: HD 60139053 0001,

TÜV Rheinland LGA Products GmbH, Nürnberg

Accessories

REF 480 ProfiGuard RDA 250

REF 481 ProfiPolish RDA 40

REF 486 ProphyRepair for cleaning and polishing, final RDA 34



Manufacturer

Alfred Becht GmbH,
Carl-Zeiss-Str. 16,
D-77656 Offenburg,
Tel.: +49 781 60586-0,
Fax.: +49 781 60586-40,
www.becht-online.de